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Positioning of rubber band on head.

Put a mask on your face. Pull the lower band over your head and position it around the neck first, and then pull the upper band over your head and position it around the back of your head.

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Making the nosepiece into a nose shape.

Make the nosepiece into a nose shape with your fingers and bring it close.

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Checking the seal of the mask and your face.

Put both hands on the mask, exhale and check for air leaks through the gap between face and mask.

If there is air leakage, adjust the position of the mask and the length of the rubber band and bring it close to the face.

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Adjusting the length of rubber band.

If a proper fit cannot be achieved, please adjust the rubber bands by following the steps below.

While lightly pressing the mask with one hand, hold the end of the rubber band with the other hand and pull it forward to adjust the length. To loosen the band, pull the rubber band that follows the contour of the face toward the ear. The rubber bands can be adjusted at four points: left, right, upper, and lower.”